Français - FR

USAGE PRÉVU

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English - EN

INTENDED USE

These *in vitro* diagnostic devices are intended for the quality control of the performances of quantitative tests listed in the value sheet.

These *in vitro* diagnostic devices are for professional use only.

COMPOSITION

- Lyophilized product prepared from human serum spiked with chemical and biological additives.
- Concentrations for each analyte to test are lot-specific.

MATERIALS REQUIRED BUT NOT PROVIDED

- VitalScientific reagents listed in the value sheet.
- General Laboratory equipment.

TRACEABILITY

The traceability is indicated on the instructions for use of the reagent used in combination with the recommended calibrator.

PRECAUTIONS FOR USE AND WARNINGS

- Consult Safety Data Sheet (SDS) for a proper handling.
- Each unit of human blood used in the manufacture of these products was tested and found to be negative/non-reactive for the presence of HbsAg, HCV and HIV1/2. The methods used were FDA approved. Nevertheless, since the risk of infection cannot be fully excluded these products must be handled as potentially infectious. In case of exposure, follow the guidelines of the competent health authorities.
- Take precautions when handling broken glass vials as sharp edges can injure the user.
- Take normal precautions and adhere to good laboratory practice.
- Use clean or single use laboratory equipment only to avoid contaminations.

WASTE MANAGEMENT

Disposal of all waste material should be in accordance with local, state and federal regulatory requirements (please refer to the Safety Data Sheet (SDS)).

PREPARATION

- Gently tap the vial so that the control material can be at the bottom of the vial.
- Carefully open the vial, avoiding the loss of lyophilizate.
- Add exactly 5 mL of distilled or deionized water.
- Carefully close the vial and dissolve the contents completely within 30 minutes by occasional gentle stirring avoiding the formation of foam.

Important: To reactivate the alkaline phosphatase, allow the reconstituted control serum to stand for one hour at 15-25 °C.

PRODUCT DETERIORATION

- The product may present a slightly hazy appearance after reconstitution. This has no effect on the performances of the product. All presence of particules would indicate deterioration.
- Do not use the product if there is visible evidence of contamination or damage (e.g. particle matter after reconstitution).
- Damage to the container may impact on product performance. Do not use the product if there is physical evidence of deterioration (e.g. leakages).

STABILITY

Prior to reconstitution :

- Store at 2-8 °C and protect from light. Do not freeze.
- Do not use after the expiry date stated on the vial label.

After reconstitution :

- These products should be immediately and tightly capped to prevent contamination and evaporation.
- Stability of the components:
Between 2 and 8 °C : 5 days
Between -25 and -15 °C : 4 weeks (when frozen once)

Exceptions:

- Stability of total bilirubin (when stored protected from light) :
Between 2 and 8 °C : 24 hours
Between -25 and -15 °C : 5 days (when frozen once)

- Stability of direct bilirubin (when stored protected from light) :
Between 2 and 8 °C : 8 hours
Between -25 and -15 °C : 5 days (when frozen once)

- Stability of alkaline phosphatase:
Between 2 and 8 °C : 24 hours
Between -25 and -15 °C : 4 weeks (when frozen once)

Note:

- Store the vials tightly capped and protected from light after reconstitution.
- Mix content thoroughly prior to use.
- Thawing frozen specimen at 37°C, and vigorous mixing (vortex), should be avoided as this may denature ferritin.

PROCEDURE

To use ELITROL I & II, follow the procedure described in the instructions for the use of the reagent used.

LIMITATIONS

Using ELITROL I & II has been validated with the VitalScientific systems (Analyzers and reagents listed in the value sheet).

Using any other system should be validated by the laboratory.

VALUES

The concentrations are indicated in the value sheet enclosed in the kit.

Individual laboratories may obtain values different from those announced. Technique, equipment and experimental error may produce slightly different values. Each laboratory should determine their own mean values.

For users of Selectra instruments allowing data import for tests, calibrators and controls, use corresponding barcode, available on the value sheet.

DECLARATION OF SERIOUS INCIDENT

Please notify the manufacturer (through your distributor) and competent authority of the Member State of the European union in which the user and/or the patient is established, of any serious incident that has occurred in relation to the device. For other jurisdictions, the declaration of serious incident should be in accordance with local, state and federal regulatory requirements. By reporting a serious incident, you provide information that can contribute to the safety of *in vitro* medical devices.

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