ESTABILIDADE

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BIBLIOGRAPHIE/BIBLIOGRAPHY BIBLIOGRAFÍA/BIBLIOGRAFIA

1. Wu, A. H. B., *Clinical guide to laboratory tests*, 4th Ed., (W.B. Saunders eds.), (2006), 306
2. Sanhai, W.R. *et al.*, *Cardiac and Muscle Disease. Clinical Chemistry: Theory Analysis, Correlation*, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 677 and appendix.
3. Apple, F. S. & Jaffe, A. S., *Cardiovascular Disease, Tietz Fundamentals of Clinical Chemistry*, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 614.
4. Schumann, G., *et al.*, *Clin Chem Lab Med.*, (2002), **40**, 635.
5. Neumeier, D., *et al.*, *Clin Chim Acta.*, (1976), **73**, 445.
6. Klein, G., *et al.*, *Clin Chem.*, (2001), **47**, Suppl. A30.
7. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, *Acta Clin Belg.*, (2004), **59**, 263.
8. Young, D.S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed., AACCPress, (1997).
9. Young, D.S., *Effects of drugs on clinical laboratory tests*, 4th Ed., AACCPress, (1995).

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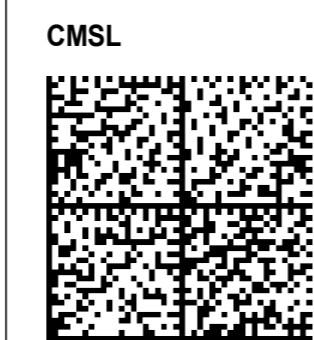
- Symbols used on our documentation are defined on ISO-15223-1 standard, except for some presented in the symbols glossary available on the VitalScientific Website. (Symbols glossary).

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- **Special Programming Instructions :** see § INSTALLATION AND USE

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English - EN

INTENDED USE

CK-MB SL /CK-MB is an *in vitro* diagnostic reagent intended for the quantitative determination of creatine kinase MB (CK-MB) in human serum and plasma samples on analyzers or semi-automatic analyzers. This *in vitro* diagnostic device is for professional use only.

CLINICAL SIGNIFICANCE ⁽¹⁻³⁾

Creatine kinase (CK) exists in 3 cytoplasmic forms: CK-MB (mostly in cardiac muscle), CK-MM (dominant isoenzyme in striated and cardiac muscles) and CK-BB (only isoenzyme in brain). Enzymatic activity of CK-MB is increased mainly during myocardial infarction, but elevated levels can be observed during muscle damage such as Duchenne dystrophy, rhabdomyolysis and trauma. In myocardial infarction, the rates of CK-MB increase quickly until reaching a peak about 24h after the onset of the infarction. The levels return to normal within 2 to 3 days. Determination of CK-MB in blood is used to reinforce the diagnostic presumption of myocardial infarction.

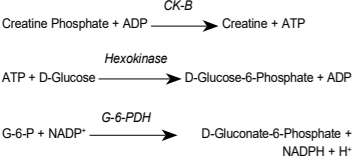
LIMITATION OF USE

CK-MB SL /CK-MB is not intended for monitoring cardiac muscle injury. The quantitative assay of CK-MB alone cannot be used to diagnose a disease or a specific pathology. The results must be interpreted in conjunction with other diagnostic test results, clinical findings and the patient's medical history.

METHOD & PRINCIPLE ^(2,4)

Immuno-inhibition/IFCC method - Kinetic.

CK-MB SL/ CK-MB reagent contains an antibody inhibiting specifically CK-M subunits (i.e. 100% of CK-MM and 50% of CK-MB). The remaining activity, corresponding to CK-B fraction activity, is quantified according to IFCC reference method for total CK measurement. CK-MB activity is then obtained by multiplying by 2 the remaining activity.

G-6-P : D-Glucose-6-Phosphate
G-6-PDH : Glucose-6-Phosphate Dehydrogenase

The rate of increase of NADPH concentration measured at 340 nm is directly proportional to the enzymatic activity.

COMPOSITION

Reagent 1: R1

Imidazole buffer, pH 6.1 (20°C)

| | | |
|--------------|---------|---------|
| D-Glucose | 25 | mmol/L |
| NADP | 2.5 | mmol/L |
| Hexokinase | ≥ 6 800 | U/L |
| Sodium azide | < 0.1 | % (w/w) |

The concentration of anti-CK-M antibody contained in reagent R1 is sufficient to inhibit 2000 U/L of CK-M at 37 °C.

Reagent 2: R2

Imidazole buffer, pH 8.9 (20°C)

| | | |
|--------------------|---------|---------|
| Creatine Phosphate | 250 | mmol/L |
| ADP | 15.2 | mmol/L |
| G-6-PDH | ≥ 8 800 | U/L |
| Sodium azide | < 0.1 | % (w/w) |

Also contains magnesium salts, *N*-acetyl cysteine, EDTA, diadenosine pentaphosphate and AMP for optimal performance.

MATERIALS REQUIRED BUT NOT PROVIDED

- CKMB-0900 CK-MB CONTROL
- Normal saline solution (NaCl 9 g/L).
- Analyzers or semi-automatic analyzers.
- General Laboratory equipment (e.g. pipette).
- Do not use materials that are not required as indicated above.

PRECAUTIONS FOR USE AND WARNINGS

- The reagents R1 and R2 are classified as hazardous (Imidazole):

DANGER May damage the unborn child. Wear protective gloves/protective clothing/eye protection/face protection. Do not handle until all safety precautions have been read and understood. IF exposed or concerned: Get medical advice/attention.

Obtain Safety data sheet (SDS) before use for a proper handling.
- The reagents contain sodium azide which may react with lead or copper plumbing to form potentially explosive metal azides. When disposing of these reagents always flush with copious amounts of water to prevent azide buildup.

- Take normal precautions and adhere to good laboratory practice.
- Use clean or single use laboratory equipment only to

avoid contamination.

- Do not interchange reagent vials from different kits.

STABILITY

Store at 2-8 °C and protect from light. Do not freeze. Do not use after expiration dates indicated on the vial labels.

PREPARATION AND STABILITY OF WORKING REAGENT

Preparation: Mix 4 volumes of reagent R1 with 1 volume of reagent R2 in an empty bottle (bottle supplied with reference CMSL-0230). This bottle must be completely emptied before refilling it with freshly prepared working reagent.

Working reagent Stability: 7 days when stored refrigerated.

These data are defined for filling volume of 9 mL in a Selectra dedicated vial of 10 mL (as supplied in reference CMSL-0230). In case of a different vial format and/or different filling volume, it is the responsibility of each laboratory to revalidate the working reagent stability.

PRODUCT DETERIORATION

- The product should be clear. Cloudiness would indicate deterioration.
- Do not use the product if there is visible evidence of contamination or damage (e.g. particle matter).
- Damage to the product container may impact on product performance. Do not use the product if there is physical evidence of deterioration (e.g. leakages or punctured container).

SAMPLES

Specimen ⁽²⁾

- Serum
- Plasma (Lithium heparin)
- Using any other specimen type should be validated by the laboratory.

Warnings and precautions

Samples should be collected in accordance with Good Laboratory Practice and appropriate guidelines that may be in place.

Storage and stability ⁽²⁾

- 48 hours at 2-8°C
- 1 month at -20°C

REFERENCE VALUES ⁽⁴⁻⁶⁾

| <i>Serum/plasma</i> | U/L | µkat/L |
|---------------------|------|--------|
| | ≤ 25 | ≤ 0.42 |

CK-MB activity must always be compared to total CK activity.

(CK-MB/ Total CK) x 100 < 6 %

The following 3 factors are indicators of damage to cardiac muscle :

| | | | |
|---------------------------------|-------|-----------|-------------------------|
| - Total CK | Men | > 171 U/L | > 2.85 µkat/L |
| | Women | > 145 U/L | > 2.41 µkat/L |
| - CK-MB | | > 25 U/L | > 0.42 µkat/L |
| - Ratio (CK-MB/ Total CK) x 100 | | | 6 - 25 % |

Note : The quoted range should serve as a guide only. It is recommended that each laboratory verifies this range or establishes a reference interval for the intended population.

INSTALLATION AND USE

For use on Selectra Pro analyzers :

- Consult operator manual.
- **Special Programming instructions: Programming special instructions is mandatory when some combinations of tests are performed together on the analyzer.** Refer to Instructions For Use of ACID SOLUTION & SYSTEM CLEANING SOLUTION for adequate programming (See PIT-SOL).

PROCEDURE

Manual Procedure

| | |
|--|--------|
| <i>Wavelength :</i> | 340 nm |
| <i>Optical path :</i> | 1 cm |
| <i>Sample/Reagent ratio :</i> | 2:25 |
| <i>Temperature:</i> | 37 °C |

Read against distilled water.

| Working reagent (4 volumes of R1 + 1 volume of R2) | 1000 µL |
|---|---------|
| Sample | 80 µL |

Mix and after 3 minute incubation, read absorbance at 30 second intervals during 5 minutes. Calculate the change of absorbance per minute (ΔA/min)

Automatic Procedure

These reagents may be used in several automatic analyzers. For Selectra Analyzers, validated applications are available on request. For Selectra TouchPro software, use the application included in the barcode available at the end of this insert.

CALCULATION

a) Total CK activity :

Determination with CK NAC SL reagent.

b) CK-MB activity :

At 340 nm, with a 1 cm light path cuvette :
Activity (U/L) = ΔA/min. x 4 286

c) Percentage of CK-MB activity in sample :

 % CK-MB =

CK-MB
Total CK

 x 100

Conversion factor : U/L x 0.0167 = µkat/L.

☞CALIBRATION

The theoretical calibration factor available in VitalScientific applications is defined from the molar absorption coefficient of NADPH.

QUALITY CONTROL

It is recommended that quality control sera such as CK-MB CONTROL be used to monitor the performance of the assay.

Controls have to be performed :

- prior to assaying patient samples,
- at least once per day,
- and/or in accordance with laboratory and regulatory requirements.
Results should be within the defined ranges. If values fall outside of the defined ranges, each laboratory should take necessary corrective measures.

WASTE MANAGEMENT

Disposal of all waste material should be in accordance with local, state and federal regulatory requirements (please refer to the Safety Data Sheet (SDS)).

PERFORMANCE DATA at 37 °C on Selectra ProM Analyzers

Performances were obtained on Selectra ProM, following CLSI technical recommendations, under controlled environmental conditions.

- Measuring range

10-600 U/L (0.17 - 10.00 µkat/L)

Samples having greater concentrations should be diluted 1:5 with NaCl 9 g/L solution and re-assayed. This procedure extends the measuring range up to 3000 U/L (50.00 µkat/L).

Do not report results outside this extended range. For users with Selectra TouchPro software, the «dilute» function performs the sample dilution automatically. Results take the dilution into account.

| Limit of Detection (LoD) and Limit of Quantification (LoQ) | | | | | |
|---|--------|---------------|--|--|--|
| LoD = | 1 U/L | (0.02 µkat/L) | | | |
| LoQ = | 10 U/L | (0.17 µkat/L) | | | |

- Precision

Imprecision data has been obtained on 2 Selectra ProM analyzers over 20 days (2 runs per day, tests performed in duplicate).

Representative results are presented below.

| | | | Mean | Within-run | Total |
|----------------|----------|------------|---------------|-------------------|--------------|
| | n | U/L | µkat/L | CV (%) | |
| Level 1 | 80 | 21 | 0.35 | 1.9 | 6.2 |
| Level 2 | 80 | 54 | 0.90 | 1.4 | 1.8 |
| Level 3 | 80 | 227 | 3.78 | 0.3 | 1.2 |

- Correlation

A comparative study has been performed between CK-MB SL/ CK-MB reagent on a Selectra ProM analyzer and a similar commercially available system on 99 human serum samples.

The sample concentrations ranged from 10 to 600 U/L (0.17-10.00 µkat/L).
The results are as follows :
Correlation coefficient : (r) = 0.999
Linear regression: y = 1.058 x - 4 U/L (0.07 µkat/L).

- Limitations/Interferences

- Hemolyzed samples should not be used since significant hemolysis may lead to falsely increased CK concentration because of adenylate kinase release.⁽¹⁾

- Studies have been performed to determine the level of interference from different compounds. The following CK-MB levels were tested: 50 U/L and 250 U/L.

No significant interference is defined by a recovery ≤±10% of the initial value.

Triglycides: No significant interference up to 1800 mg/dL (20.3 mmol/L).
Unconjugated bilirubin: No significant interference up to 30.0 mg/dL (513 µmol/L).

Conjugated bilirubin: No significant interference up to 29.5 mg/dL (505 µmol/L).

Ascorbic acid: No significant interference up to 20.0 mg/dL.

Acetaminophen: No significant interference up to 30 mg/dL.

Acetylsalicylic acid: No significant interference up to 200 mg/dL.

- In very rare cases, monoclonal gammopathies (multiple myeloma), in particular IgM type (Waldenström's macroglobulinemia) can cause unreliable results.⁽⁷⁾

- Many other substances and drugs may interfere. Some of them are listed in reviews published by Young.⁽⁸⁻⁹⁾

- On board stability

Refer to § PREPARATION AND STABILITY OF WORKING REAGENT.

These performances have been obtained using Selectra ProM analyzer. Results may vary if a different instrument or a manual procedure is used.

☞The performances of applications not validated by VitalScientific are not warranted and must be defined by the user.

DECLARATION OF SERIOUS INCIDENT

Please notify the manufacturer (through your distributor) and competent authority of the Member State of the european union in which the user and/or the patient is established, of any serious incident that has occurred in relation to the device. For other jurisdictions, the declaration of serious incident should be in accordance with local, state and federal regulatory requirements. By reporting a serious incident, you provide information that can contribute to the safety of *in vitro* medical devices.

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