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BIBLIOGRAPHIE/BIBLIOGRAPHY BIBLIOGRAFIA/BIBLIOGRAFIA

- Dufour D. R., *The Liver: Function and Chemical Pathology. Clinical Chemistry: Theory, Analysis, Correlation*, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds), (2010), 586 and appendix.
- Wu, A.H.B., *Clinical guide to laboratory tests*, 4th Ed., (W.B. Saunders eds.), (2006), 916.
- Doumas, B.T., et al., *Clin. Chem.*, (1981), 27, 1642.
- Guder, W.G., et al., *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*.(2002). WHO/DILU/LAB/99.1 Rev.2.
- Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, *Acta Clin Belg.*, (2004), 59, 263.
- Young, D.S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed., AACC Press (1997).
- Young D.S., *Effects of drugs on clinical laboratory tests*, 4th Ed., AACC Press (1995).

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- Symbols used on our documentation are defined on ISO-15223-1 standard, except for some presented in the symbols glossary available on the VitalScientific Website. (Symbols glossary).

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English - EN

INTENDED USE

TOTAL PROTEIN PLUS is an *in vitro* diagnostic reagent intended for the quantitative determination of total protein in human serum and plasma samples on analyzers or semi-automatic analyzers.
The standard is intended for the calibration of the reagent.
These *in vitro* diagnostic devices are for professional use only.

CLINICAL SIGNIFICANCE ⁽¹⁻²⁾

Plasma proteins are mainly albumin and globulins. Most plasma proteins are synthesized by the liver, except immunoglobulins.
Many factors can modify proteinemia. Increase of the plasmatic volume (salt retention, hyperhydration) or its reduction (dehydration) induce mild hypoproteinemia or mild hyperproteinemia, respectively.
For a normal plasmatic volume, hypoproteinemia can occur in severe protein insufficiencies (malabsorption, maldigestion, dietary insufficiency), or renal (increased protein loss) and hepatic diseases (impaired protein synthesis). Hyperproteinemia can reveal hyperimmunoglobulinemia (multiple myeloma, infection).
In practice, total proteins measurement is mostly indicated to help diagnose liver and kidney disorders, or to determine nutritional status.


LIMITATION OF USE

The quantitative assay of analyte alone cannot be used to diagnose a disease or a specific pathology.
The results must be interpreted in conjunction with other diagnostic test results, clinical findings and the patient's medical history.

METHOD & PRINCIPLE ⁽³⁾

Biuret. End point

Serum proteins form a coloured complex in the presence of copper (II) salt in alkaline solution.

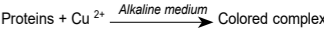COMPOSITION

Reagent: R
Copper sulfate 6 mmol/L
Sodium hydroxide 490 mmol/L
Also contains iodide and tartrate salts for optimal performance.
Standard: Std (Ref : PROB-0600/0700)
Albumin 6 g/dL
Sodium azide < 0.1 % (w/w)